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RULES FOR THE SCOOTER

- The maximum load capacity is 115 kg.
 - Before turning or negotiating curves, reduce speed.
 - Turn off the scooter before getting on, off, or changing any accessories or components.
 - Do not ride on slopes with a slope greater than 8 degrees, and be very careful when negotiating curves or turning on an incline.
 - Before riding the scooter, make sure the seat and handlebars are firmly secured and in the correct position for riding.
 - **Ride at a low speed in areas with uphill's, downhills, ramps or uneven terrain. do not ride on sand or grassy lawns. Do not ride down steps or sidewalks with the scooter in electric mode. But make sure it is in manual mode and push it gently to go down sidewalks if you don't have handicap descent.**
 - You can go over obstacles with a maximum height of 15 mm, but do not try to go over them in sloping areas or where there are edges.
 - The scooter can pass a hole with a maximum length of 10cm.
 - Do not attempt to reverse on slopes.
 - The maximum capacity of the basket (accessory) is 3kg.
- Follow all instructions and remember to return it clean.
- Have a good trip and enjoy yourself
- And if you have any difficulties, call us at the following number: 3926509237 (also Whatsapp)

TECHNICAL SPECIFICATIONS:

- **Total length:** 1000 mm.
- **Total width:** 510 mm.
- **Total height:** 890 mm.
- **Front and rear wheels:** 195 mm diameter.
- **Weight with battery:** 40 kg.
- **Maximum speed:** 6.4 km/h.
- **Maximum load capacity:** 115 kg.
- **Ground clearance:** 30 mm.
- **Maximum surmountable slope:** 8 degrees.
- **Slope curve:** 35 mm / 1.4".
- **Curve radius:** 1270 mm.
- **Brake:** electromechanical.
- **Seat type:** padded folding.
- **Seat width:** 425 mm.

- **Motor:** 250W, 4700 r.p.m.
- **Battery:** (2) 12V. 12Ah.
- **Battery weight:** 9.2 kg.
- **Autonomy:** 10 km.
- **Battery charge:** 1.8A outside the vehicle.
- **Electronics:** On/off key, battery level indicator, speed control knob.

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